

SEXUAL HEALTH COLLECTION KIT

A vaginal swab sample collection kit used for the collection and transportation of samples for laboratory analysis of sexually transmitted infections by nucleic acid technique (NAT)

This kit contains the materials required for either sample self-collection or collection by a health/social care professional.

IVD For *in vitro* diagnostic use, sample self-collection

REF KT357

**UK
CA**

i Carefully read the enclosed instruction leaflet before use

! Important: Do not use kit if security seal is broken. Keep out of reach of children

⌘ Do not re-use


18+ For use by persons aged 18 and over

Sample collection instructions (Steps 1-14)

- Please ensure the kit is within expiry date and read these instructions carefully and completely before attempting to collect the sample.
- If your kit requires online activation, please follow the instructions provided by your healthcare organisation. If you need assistance please contact the healthcare organisation who arranged the test.
- For alternative language versions of these kit instructions please visit www.tdlpathology.com

Your sample collection kit contents


Please check that the kit contains all of the items outlined below. Do not proceed with sample taking if any items are missing, contact the healthcare organisation who arranged your test for assistance.

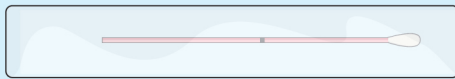

Sample collection swab, CE **STERILE** EO



Swab specimen transport tube, **IVD**

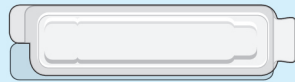

Protective packing wallet



Sample label



Return security label



Return postal envelope



Collection kit outer box

VAGINAL SWAB SAMPLE COLLECTION

- 1** If not already pre-populated please complete the test request form, provided by your doctor/clinic, separate to this kit. **Make sure the sample date is completed.**



- 2** Clearly complete the sample collection tube label using a ball point pen with:

- Your First name
- Your Surname
- Your Date of birth
- Sample type: Vaginal



3 IMPORTANT SAMPLE TRACKING INFORMATION!


The enclosed postal envelope has a postal tracking label applied. Please make a note of the unique tracking number displayed under the barcode on the postal tracking label, as shown below. Keep this in a safe place for future reference. **You can track delivery of your Royal Mail parcel by entering this number using the following link:** <http://www.royalmail.com/track-your-item>




Write your tracking number here for reference – keep this safe.




Wash hands using soap and water and dry them.

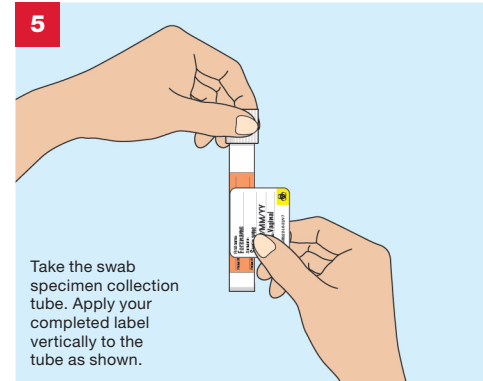


Take the swab specimen collection tube. Apply your completed label vertically to the tube as shown.

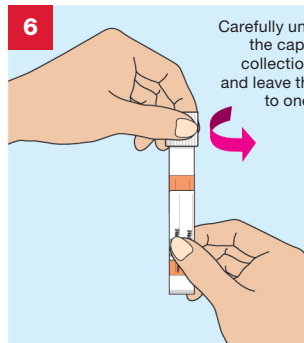


Carefully unscrew the cap of the collection tube and leave the cap to one side.

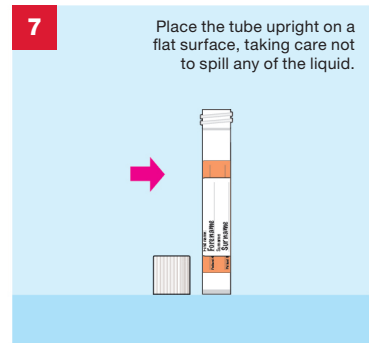


Place the tube upright on a flat surface, taking care not to spill any of the liquid.

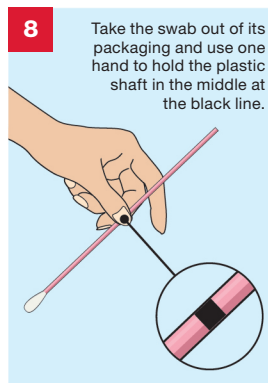


8 Take the swab out of its packaging and use one hand to hold the plastic shaft in the middle at the black line.

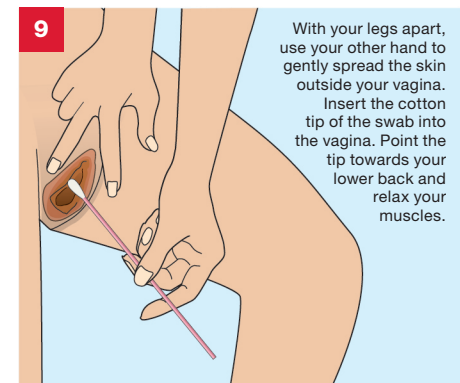

9 With your legs apart, use your other hand to gently spread the skin outside your vagina. Insert the cotton tip of the swab into the vagina. Point the tip towards your lower back and relax your muscles.

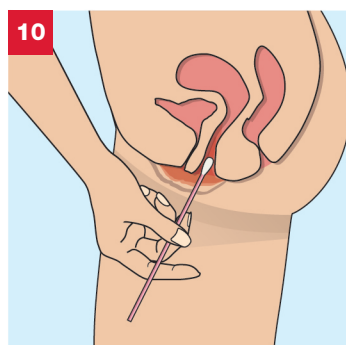


10 Gently insert the swab about 5cm into your vagina. Your fingers on the black line will stop you going in too far. If the swab does not slide easily, gently rotate it as you put it in. If it is too difficult, do not attempt to continue.

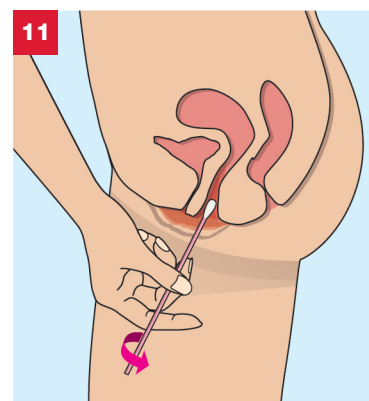

11 Rotate the swab for 5 seconds (count to 5 slowly), making sure it touches the walls of your vagina. Carefully pull out the swab.


12

Carefully put the swab into the collection tube. Holding the tube steady, snap the stick off at the black line.

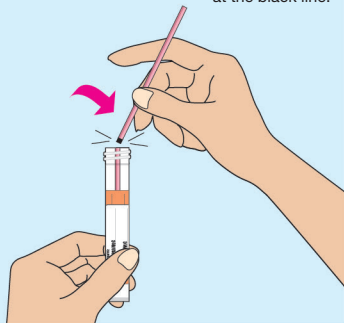

13

Discard the end of the swab in a bin. Screw the cap back onto the tube tightly. Place the tube into the protective packing wallet and firmly close.

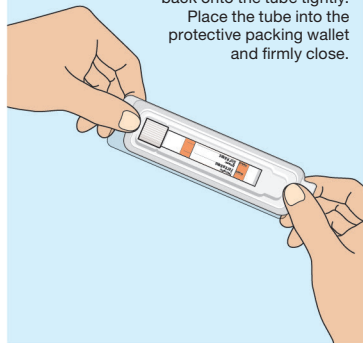

14 IMPORTANT CHECKLIST

Before you return your sample please do the following:

- Place your completed request form supplied by your doctor/clinic into the postal envelope
- Make sure the swab specimen transport tube is labelled and is placed inside the protective packing wallet
- Place the protective packing wallet into the collection kit outer box
- Close the collection kit outer box firmly and apply the return security label to seal it
- Place the collection kit outer box into the return postal envelope
- Check that you have taken note of your postal tracking number

You are now ready to seal the postal envelope.
Please store at room temperature until posted.

Please post your sample to **The Doctors Laboratory** as soon as possible (ideally on same day or within 24 hours of sample collection) from **ANY** Royal Mail post box in the UK. No stamp is required within the UK.

If you need assistance please contact The Doctors Laboratory on 020 7307 7373 or email samples@tdlpathology.com.

Warnings and precautions

- This kit is designed for use by persons aged 18 and over and upon request of a healthcare professional or healthcare organisation.
- The kit should not be used by individuals lacking the physical or mental capacity to correctly follow the self-collection instructions. If you have problems please consult with your advising healthcare professional.
- If you have experienced recent pelvic pain, pain during or as a result of sexual intercourse and/or unusual vaginal discharge or odor please discuss this with your healthcare professional before collecting your sample.
- The Hologic transport buffer contains no substances which at their given concentration, are considered to be hazardous to health. In the case of contact with eyes, skin or mouth please and rinse thoroughly with clean water, if ingested please drink plenty of water and if any symptoms occur please consult a medical professional. In the case of spillages please wipe with absorbent paper and dispose in normal household waste. For further information please refer to safety datasheets which can be found at www.hologic.com or contact MolecularSupport@hologic.com.
- Samples arriving at the laboratory which show evidence of the below may not be tested:
 - Leaking sample/no liquid present in collection tube
 - No swab inside the collection tube
 - Two or more swabs inside the collection tube
 - Swab placed upside down in the collection tube
 - More than 1 collection tube per return box
 - Different swab type/brand to the one supplied
 - General damage
- The sample collection swab has a score line, which after sampling helps the swab break at the correct position when pressure is applied to allow it to fit in the collection tube. If this score line is broken, appears damaged or the swab arrives in two broken parts, do not use and request another kit from your healthcare organisation.
- Use with caution if allergic to nylon fibre and ABS (Acrylonitrile butadiene styrene) material. If any problems arise during the sample collection process please contact your healthcare organisation.
- The accuracy of your results may be compromised if you do not read and follow the instructions in full.
- Samples arriving at the laboratory which show signs of degradation or general damage or arrive after 60 days (Aptima swab) of sample taking may not be tested.

Materials required but not provided

- **Test request form**
This will be provided by your healthcare professional or healthcare organisation.
Please complete the request form with your details as instructed.

Request Form	
First Name	_____
Surname	_____
DOB	_____
Sex	_____
EXAMPLE	

Laboratory Tests

- The tests and procedures undertaken by The Doctors Laboratory Limited are verified and performed in line with supplier product instructions for use and supported by additional validation data for use with self-collected capillary blood.
- Test results are provided in line with clinically approved results pathways, agreed between The Doctors Laboratory Limited and the patients designated healthcare professional or healthcare organisation.

KEY TO SYMBOLS



In vitro diagnostic medical device



Use by end of year-month



UK Conformity Assessed



Catalogue number



Temperature limits: 4°–25°C



EU Conformity Assessed



Batch code



Caution, consult accompanying documents



Do not re-use



Sterilized using ethylene oxide



Keep away from sunlight



Consult instructions for use



Biological risk. Biological sample, treat as potentially infectious



Keep dry



For use by persons aged 18 and over



Do not use if package is damaged



Manufacturer



MedDX Services Ltd

Unit 2 Newtons Court, Crossways Business Park
Dartford, Kent, DA2 6QL, UK

E-mail: support@mdxsvc.com Website: www.mdxsvc.com

The Doctors Laboratory Limited

Distribution is restricted to regions in which the product is registered by the manufacturer.

© The Doctors Laboratory Limited, 2022. Illustrations: © Jag Matharu/Thin Air Productions

RM1252/V1, TAP4947/17-05-22/V12