





SEXUAL HEALTH COLLECTION KIT

A vaginal and extra-genital (throat and rectal) swab sample collection kit used for the collection and transportation of samples for laboratory analysis of sexually transmitted infections by nucleic acid technique (NAT)

This kit contains the materials required for either sample self-collection or collection by a health/social care professional.


IVD For <i>in vitro</i> diagnostic use, sample self-collection	REF KT428	UK CA
 Carefully read the enclosed instruction leaflet before use	 Important: Do not use kit if security seal is broken. Keep out of reach of children	
 Do not re-use	 For use by persons aged 18 and over	

Sample collection instructions (Steps 1–29)

- Please ensure the kit is within expiry date and read these instructions carefully and completely before attempting to collect the sample.
- If your kit requires online activation, please follow the instructions provided by your healthcare organisation. If you need assistance please contact the healthcare organisation who arranged the test.
- For alternative language versions of these kit instructions please visit www.tdlpathology.com

Your sample collection kit contents

Please check that the kit contains all of the items outlined below. Do not proceed with sample taking if any items are missing, contact the healthcare organisation who arranged your test for assistance.

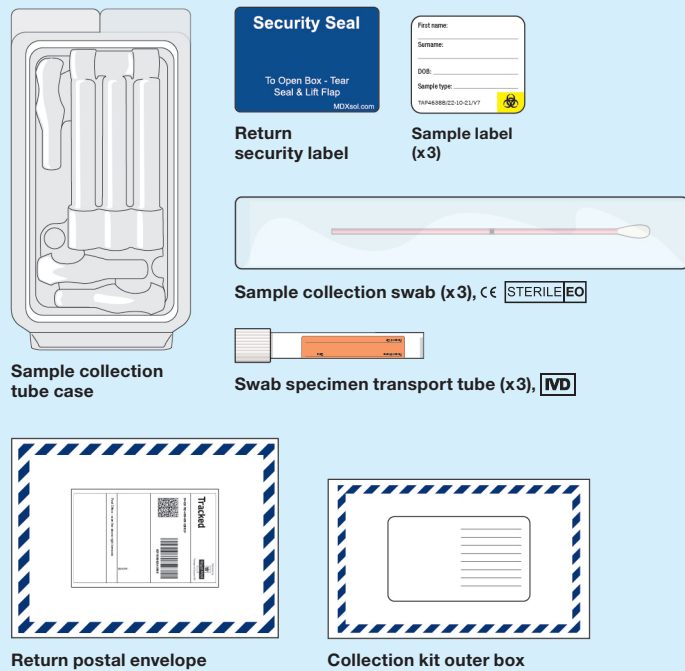

Security Seal
To Open Box – Tear Seal & Lift Flap
MDX001.com

Return security label

Sample label (x3)
First name: _____
Surname: _____
DOB: _____
Sample type: _____

Sample collection swab (x3), CE STERILE EO

Swab specimen transport tube (x3), IVD


Sample collection tube case

Return postal envelope

Collection kit outer box

THROAT SWAB SAMPLE COLLECTION

1 If not already pre-populated please complete the test request form, provided by your doctor/clinic, separate to this kit. **Make sure the sample date is completed.**

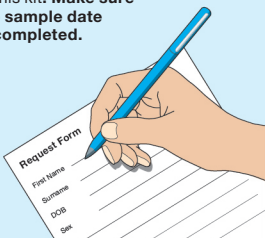

2 Clearly complete a sample collection tube label using a ball point pen with:

- Your First name
- Your Surname
- Your Date of birth
- Sample type: Throat



3 IMPORTANT SAMPLE TRACKING INFORMATION!

The enclosed postal envelope has a postal tracking label applied. Please make a note of the unique tracking number displayed under the barcode on the postal tracking label, as shown below. Keep this in a safe place for future reference. **You can track delivery of your Royal Mail parcel by entering this number using the following link:** <http://www.royalmail.com/track-your-item>




Write your tracking number here for reference – keep this safe.



4


Wash hands using soap and water and dry them.

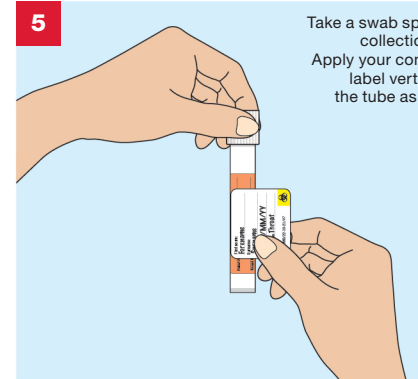

5


Take a swab specimen collection tube. Apply your completed label vertically to the tube as shown.

6 The sample collection tube case has been designed to provide a stable way of holding the swab collection tube whilst a sample is taken. Please insert the tube as shown, remove the cap and put it to one side whilst you take your sample. Take care not to spill any of the liquid the tube contains.

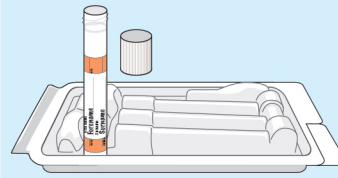

Do not touch or pierce the foil top of the lid.


7 Take the swab out of its packaging and use one hand to hold the plastic shaft in the middle at the black line.

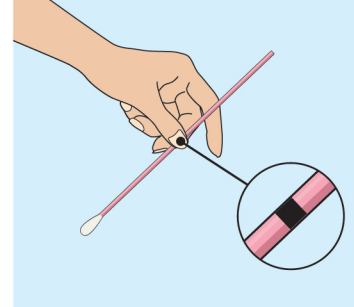


8 Carefully insert the swab into the mouth ensuring contact with the 5 key areas:

- 1 Tonsil
- 2 Posterior wall
- 3 Uvula
- 4 Posterior wall
- 5 Tonsil

Please do not let the end of the swab touch other areas of the mouth.

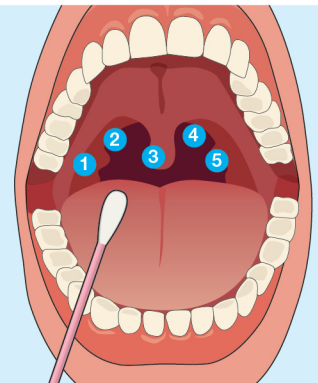


9 Carefully put the swab into the collection tube. Holding the tube steady, snap the stick off at the black line.

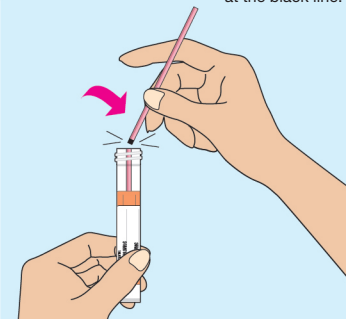

10 Discard the stick end of the swab in a bin leaving the cotton end in the liquid. Screw the cap back onto the tube tightly.

Place the tube into the protective packing case and firmly close.

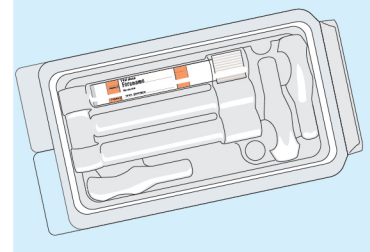

VAGINAL SWAB SAMPLE COLLECTION

11

Clearly complete the sample collection tube label using a ball point pen with:

- Your First name
- Your Surname
- Your Date of birth
- Sample type: Vaginal




12

Wash hands using soap and water and dry them.



23


Take a swab specimen collection tube. Apply your completed label vertically to the tube as shown.

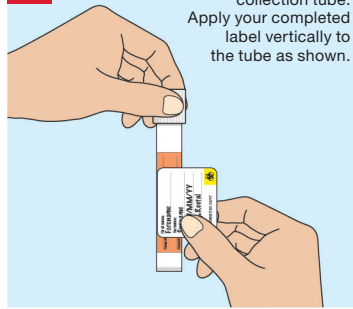

24


The sample collection tube case has been designed to provide a stable way of holding the swab collection tube whilst a sample is taken. Please insert the tube as shown, remove the cap and put it to one side whilst you take your sample. Take care not to spill any of the liquid the tube contains.

Do not touch or pierce the foil top of the lid.

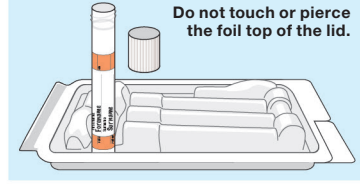

13


Take a swab specimen collection tube. Apply your completed label vertically to the tube as shown.

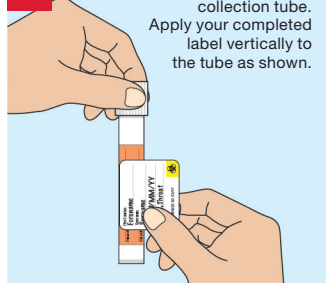

14


The sample collection tube case has been designed to provide a stable way of holding the swab collection tube whilst a sample is taken. Please insert the tube as shown, remove the cap and put it to one side whilst you take your sample. Take care not to spill any of the liquid the tube contains.

Do not touch or pierce the foil top of the lid.

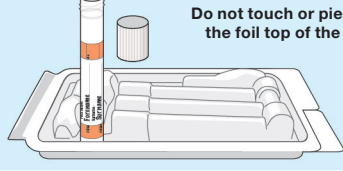


25

Take the swab out of its packaging and use one hand to hold the plastic shaft in the middle at the black line.

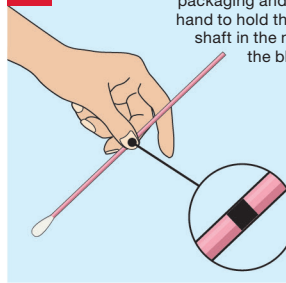


26

Holding the swab tightly, slowly insert the tip into your anus. Twist the swab slightly as you slowly pull it out of your anus.

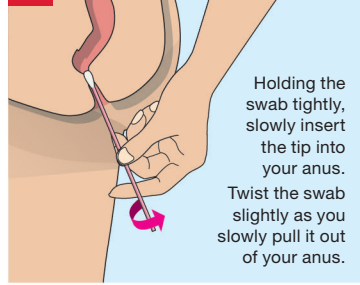


15

Take the swab out of its packaging and use one hand to hold the plastic shaft in the middle at the black line.

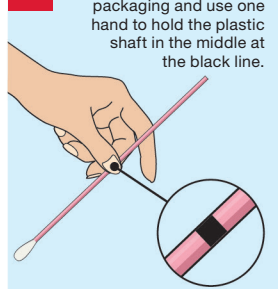


16

With your legs apart, use your other hand to gently spread the skin outside your vagina. Insert the cotton tip of the swab into the vagina. Point the tip towards your lower back and relax your muscles.

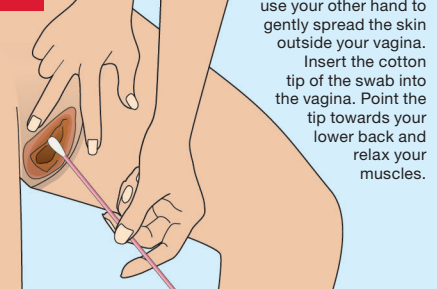


27

Carefully put the swab into the collection tube. Holding the tube steady, snap the stick off at the black line.

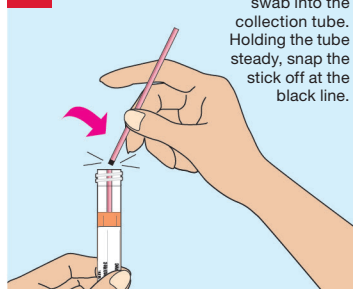


28

Discard the stick end of the swab in a bin leaving the cotton end in the liquid. Screw the cap back onto the tube tightly. Place the tube into the protective packing case and firmly close.

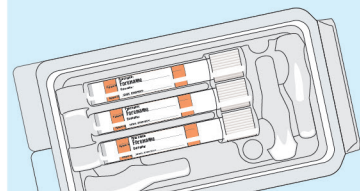


17

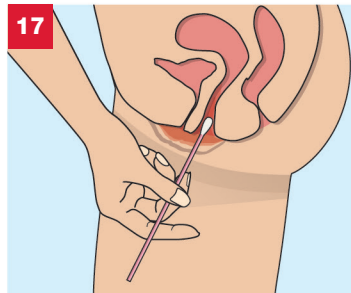

Gently insert the swab about 5cm into your vagina. Your fingers on the black line will stop you going in too far. If the swab does not slide easily, gently rotate it as you put it in. If it is too difficult, do not attempt to continue.


18

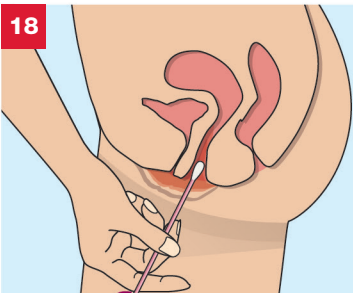

Rotate the swab for 5 seconds (count to 5 slowly), making sure it touches the walls of your vagina. Carefully pull out the swab.


19

Carefully put the swab into the collection tube. Holding the tube steady, snap the stick off at the black line.

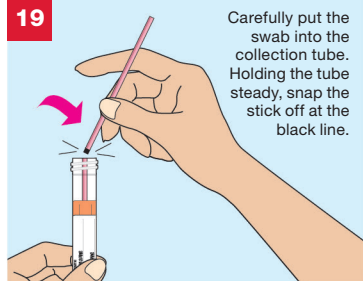

20

Discard the stick end of the swab in a bin leaving the cotton end in the liquid. Screw the cap back onto the tube tightly. Place the tube into the protective packing case and firmly close.

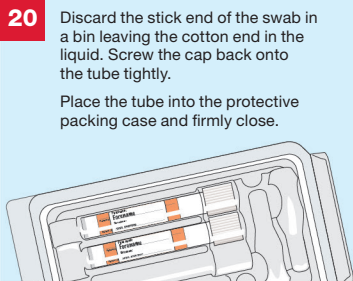

29 IMPORTANT CHECKLIST

Before you return your sample please do the following:

- Place your completed request form supplied by your doctor/clinic into the postal envelope
- Make sure each swab specimen transport tube is labelled and is placed inside the protective packing case
- Place the protective case into the collection kit outer box
- Close the collection kit outer box firmly and apply the return security label to seal it
- Place the collection kit outer box into the return postal envelope
- Check that you have taken note of your postal tracking number

You are now ready to seal the postal envelope. Please store at room temperature until posted.

Please post your sample to **The Doctors Laboratory** as soon as possible (ideally on same day or within 24 hours of sample collection) from **ANY** Royal Mail post box in the UK. No stamp is required within the UK.


If you need assistance please contact The Doctors Laboratory on 020 7307 7373 or email samples@tdlpathology.com.

RECTAL SWAB SAMPLE COLLECTION

21

Clearly complete the sample collection tube label using a ball point pen with:

- Your First name
- Your Surname
- Your Date of birth
- Sample type: Rectal

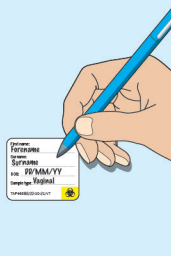

22

Wash hands using soap and water and dry them.



Warnings and precautions

- This kit is designed for use by persons aged 18 and over and upon request of a healthcare professional or healthcare organisation.
- The kit should not be used by individuals lacking the physical or mental capacity to correctly follow the self-collection instructions. If you have problems please consult with your advising healthcare professional.
- If you have experienced recent pelvic pain, pain during or as a result of sexual intercourse and/or unusual vaginal discharge or odor please discuss this with your healthcare professional before collecting your sample.
- The Hologic transport buffer contains no substances which at their given concentration, are considered to be hazardous to health. In the case of contact with eyes, skin or mouth please and rinse thoroughly with clean water, if ingested please drink plenty of water and if any symptoms occur please consult a medical professional. In the case of spillages please wipe with absorbent paper and dispose in normal household waste. For further information please refer to safety datasheets which can be found at www.hologic.com or contact MolecularSupport@hologic.com.
- Samples arriving at the laboratory which show evidence of the below may not be tested:
 - Leaking sample/no liquid present in collection tube
 - No swab inside the collection tube
 - Two or more swabs inside the collection tube
 - Swab placed upside down in the collection tube
 - Different swab type/brand to the one supplied
 - General damage
- The sample collection swab has a score line, which after sampling helps the swab break at the correct position when pressure is applied to allow it to fit in the collection tube. If this score line is broken, appears damaged or the swab arrives in two broken parts, do not use and request another kit from your healthcare organisation.
- Use with caution if allergic to nylon fibre and ABS (Acrylonitrile butadiene styrene) material. If any problems arise during the sample collection process please contact your healthcare organisation.
- The accuracy of your results may be compromised if you do not read and follow the instructions in full.
- Samples arriving at the laboratory which show signs of degradation or general damage or arrive after 60 days (Aptima swab) of sample taking may not be tested.
- Please note that some medicines or medicinal products may be considered interfering substances for certain biochemical or immunoassay investigations. Please consider the potential impact of interfering substances when interpreting results.
- Where appropriate, out of range, abnormal or positive test results deemed clinically significant should be confirmed with a confirmatory venous sample.

Materials required but not provided

• Test request form

This will be provided by your healthcare professional or healthcare organisation.

Please complete the request form with your details as instructed.

• Clean tissue paper

Request Form
Name _____
Address _____
Date _____
Sex _____
EXAMPLE

Laboratory Tests

- The tests and procedures undertaken by The Doctors Laboratory Limited are verified and performed in line with supplier product instructions for use and supported by additional validation data for use with self-collected capillary blood.
- Test results are provided in line with clinically approved results pathways, agreed between The Doctors Laboratory Limited and the patients designated healthcare professional or healthcare organisation.

KEY TO SYMBOLS

IVD	For in vitro diagnostic use	Use by end of year-month	UKCA	UK Conformity Assessed
REF	Catalogue number	Manufacturer	CE	Conformity marking EC Directive 98/79/EC
LOT	Batch code	Temperature limits: 4°–25°C	No re-use	
STERILE EO	Sterilized using ethylene oxide	Caution, consult accompanying documents	Not made with natural rubber latex	
Biological risk. Biological sample, treat as potentially infectious		Keep away from sunlight	Consult instructions for use	
Keep dry		For use by persons aged 18 and over		



MedDX Services Ltd

Unit 2 Newtons Court, Crossways Business Park
Dartford, Kent, DA2 6QL, UK

E-mail: support@mdxsvc.com Website: www.mdxsvc.com

The Doctors Laboratory Limited

Distribution is restricted to regions in which the product is registered by the manufacturer.

© The Doctors Laboratory Limited, 2022. Illustrations: © Jag Matharu/Thin Air Productions
RM1600/V1, TAP5109/25-08-22/V4

